



# Confidence in RIP

## Pro-Tech zRIP DuraBelt

The double-adjust Pro-Tech zRIP DuraBelt effort sensor is designed to help sleep lab professionals acquire reliable, clear, and accurate respiratory signals all night long. The sensor belts have been designed for complete thoracic and abdominal coverage while providing true breathing movement tracings with no false paradoxing.

- Dual-sided belts with new design modifications are intended to make it easier for you to do initial placement, and middle-of-the-night adjustments, on patients
- Tested with additional cleaning options to help comply with your facility's protocol
- A robust buckle design integrates the jack/connection in one solid interface to help minimize strain on the woven sensor and enhance durability
- The enhanced woven wire is designed to increase the durability of the sensor during normal use and cleaning

**PHILIPS**

**RESPIRONICS**

sense **and** simplicity

### Cleaning instructions for the zRIP DuraBelt effort sensor belts

- The preferred cleaning method is to wash the belts using warm water and household laundry detergent, and then air dry.
- They may be cleaned with Amphyl Hospital Bulk Disinfectant Cleaner, DisCide ULTRA Disinfectant, and CaviCide when used according to the manufacturer's instructions.
- In normal use conditions, the zRIP DuraBelt exceeds the current non-critical item recommendation by the Centers for Disease Control and Prevention (CDC).
- Provides more cleaning options than comparable competitor sensors.

### Performance results

DuraBelt performed 5,658,000 stretches from 120% to 200% of resting position after environmental, cleaning, and disinfection testing. There is an average of 5,760 breaths stretching an effort belt during an eight-hour study.



zRIP DuraBelt thoracic and abdomen belts



zRIP DuraBelt thoracic and abdomen belts with standard zRIP module

### How to reach us

Call SleepVIP at 1-800-345-6443 (toll free, US only), or your local Philips Respironics sales representative. See product information at: [www.zripsensors.respironics.com](http://www.zripsensors.respironics.com).

All of our products are backed by our market-leading service and support. In a recent survey, sleep professionals voted Philips Respironics as the most recommended diagnostics supplier in the U.S.

Respironics, Pro-Tech, DuraBelt, and zRIP are trademarks of Respironics, Inc. and its affiliates. All rights reserved. All other trademarks are the property of their respective owners.

### zRIP DuraBelt kit part numbers

Old #	New #	
P1808	1076329	zRIP DuraBelt Kit, Adult
P1809	1076330	zRIP DuraBelt Kit, Pediatric
P1837	1075969	zRIP DuraBelt Kit, Adult, PDx
P1818	1076333	zRIP DuraBelt Kit, Adult, A5
P1819	1076334	zRIP DuraBelt Kit, Pediatric, A5
P1839	1076336	zRIP DuraBelt Sum RT Kit, Adult
P1840	1076337	zRIP DuraBelt Sum RT Kit, Pediatric
P1812	1076331	zRIP DuraBelt Kit, Adult, A3
P1815	1076332	zRIP DuraBelt Kit, Pediatric, A3
P1841	1075968	zRIP DuraBelt Kit, Adult, QDC
P1842	1075967	zRIP DuraBelt Kit, Pediatric, QDC
P1820	1076335	zRIP DuraBelt Kit, Adult, DC
P1821	1075972	zRIP DuraBelt Kit, Pediatric, DC

### zRIP DuraBelt effort sensors

P1806	1073622	zRIP DuraBelt, Adult, 24" - 74"
P1822	1075973	zRIP DuraBelt, Adult, Large, 45" - 123"
P1807	1075970	zRIP DuraBelt, belt only, Pediatric, 16" - 42"

### zRIP modules

P1732	n/a	zRIP module
P1753	n/a	zRIP Sum RT module
P1736R	n/a	zRIP module, Alice 5
P1734	n/a	zRIP module, Alice 3
P1738	n/a	zRIP module, DC (Bio-logic)

### zRIP wiresets

P1133	n/a	zRIP wireset
P1137	n/a	zRIP wireset, Medicare
P1170	n/a	zRIP wireset, Alice PDx
P1176	n/a	zRIP cable abdomen, QDC
P1177	n/a	zRIP cable ribcage, QDC



©2010 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.

SB 11/23/10 MCI 4103794

Philips Healthcare is part of Royal Philips Electronics

Respironics Asia Pacific  
+65 6298 1088  
Respironics Australia  
+61 (2) 9666 4444  
Respironics Europe, Middle East, Africa  
+33 1 47 52 30 00  
Respironics United Kingdom  
+44 800 1300 845

Philips Respironics  
1010 Murry Ridge Lane  
Murrysville, PA 15668

Customer Service  
+1 724 387 4000  
+1 800 345 6443 (toll free, US only)  
[www.philips.com/respironics](http://www.philips.com/respironics)